



1stQ E-IFU

Confidentiality Statement

This document contains confidential information, and as such may not be disclosed to 3rd party without the permission of 1stQ GmbH. All rights reserved.

1STQ ADDON INTRAOCULAR LENSES

INSTRUCTION FOR USE

EN


MODELS

| Art.-No. | Material | Design |
|-----------------------------------|-------------|----------------------------|
| A4SW00 | Hydrophilic | Monofocal refractive |
| A4EW0M, A4DW0M A4EW0K, A4DW0K | Hydrophilic | Trifocal diffractive |
| A4TW0T, A4TW00, A4FW0T, A4FW00 | Hydrophilic | Monofocal refractive toric |
| A4HW0M | Hydrophilic | Trifocal diffractive toric |
| A4MW00 | Hydrophilic | Multifocal refractive |
| A4EW0H, A4DW0H | Hydrophilic | EDOF |

DESCRIPTION

1stQ AddOn intraocular lenses are single piece, sterile, foldable, hydrophilic acrylic optical devices with UV-absorbent. Different models are controlled individually for their optical and mechanical properties.

Graph 1: Average spectral transmittance of 1stQ IOLs

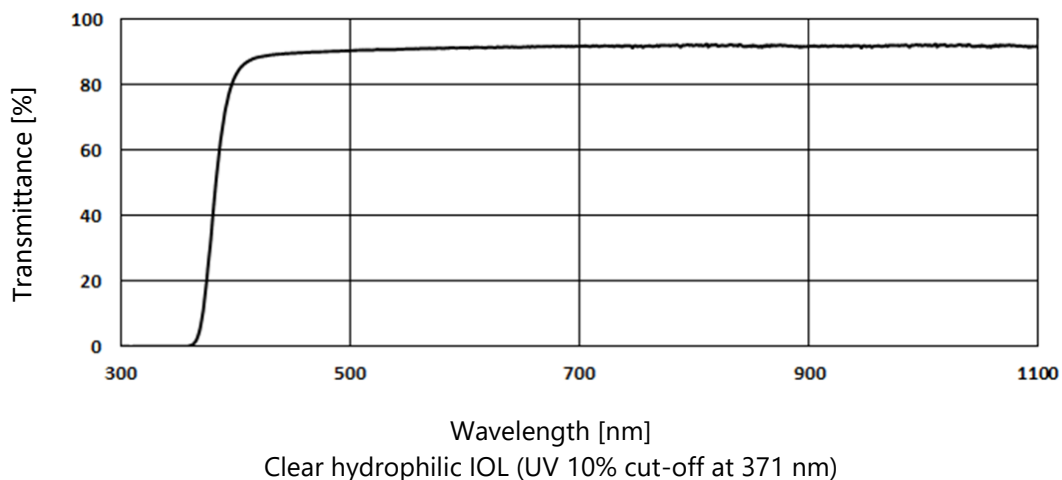

TORIC MODELS

In case of monotoric lenses the toric surface is on the anterior side, whereas in case of bitoric lenses both sides are toric.

TRIFOCAL MODELS

The anterior surface is the diffractive side of the lens. The added power for near vision is indicated on the label.

DEVICES INTENDED FOR USE TOGETHER WITH THE IOL

The IOL should be implanted with a suitable injector. A compatibility chart can be found on our website: www.1stq.eu/compatibility. Devices other than those listed in the chart have not been tested and cannot be recommended.

PACKAGING

The hydrophilic lenses are supplied steam sterilized in a container filled with sterile water. The containers are packed in a protective blister.

EXPIRATION DATE

1stQ Intraocular lenses are sterile unless their primary packaging is damaged. The expiry date is printed on the labels of the outer packaging and the protective blister or peel-pouch. Do not use an IOL after its expiry date.

INTENDED PURPOSE

1stQ Addon Intraocular lenses are posterior chamber intraocular lenses intended for implantation into the ciliary sulcus in the posterior chamber of pseudophakic patients with a primary intraocular lens implanted in the capsular bag.

MEDICAL INDICATION

Monofocal Models

A4SW00 is indicated for pseudophakic patients with a primary IOL in the capsular bag who aspire to have a refractive adjustment

Trifocal diffractive Models

A4EW0M, A4DW0M, A4EW0K, A4DW0K are indicated for pseudophakic patients with a primary IOL in the capsular bag who aspire to have additional near and intermediate vision with increased spectacle independence and optionally a refractive adjustment.

A4EW0H, A4DW0H are indicated for pseudophakic patients with a primary IOL in the capsular bag who aspire to have additional intermediate with increased spectacle independence and optionally a refractive adjustment.

Monofocal toric Models

A4TW0T, A4TW00, A4FW0T, A4FW00 are indicated for pseudophakic patients with a primary IOL in the capsular bag who aspire to have a correction of their corneal astigmatism and optionally a refractive adjustment.

Diffractive toric Models

A4HW0M is indicated for pseudophakic patients with a primary IOL in the capsular bag who aspire to have improved near and intermediate vision with increased spectacle independence including a correction of corneal astigmatism and optionally refractive adjustment.

SML Scharioth Macula Lense

A4MW00 is indicated for pseudophakic patients with a dry form of age-related macular degeneration having a primary IOL in the capsular bag who aspire to have improved near vision.

CONTRAINDICATIONS

There are no known contraindications to the use of 1stQ AddOn IOL when used as recommended.

PATIENT TARGET GROUP

Pseudophakic adult patients (18 years old and older) with a primary capsular bag fixated intraocular lens.

INTENDED USERS

1stQ Addon Intraocular lenses must be handled and implanted by a qualified and properly trained ophthalmic surgeon.

PRECAUTIONS

The safety and effectiveness of 1stQ Intraocular lenses have not been studied in patients with certain existing conditions and /or intraoperative complications listed below (as these patients were excluded from clinical studies). Careful preoperative and perioperative evaluation and clinical judgement should be made by the ophthalmic surgeon to decide the risk/benefit ratio before the implantation in the following (non-exhaustive) pre-existing conditions:

- Aphakia
- Microphthalmia
- Shallow anterior chamber (< 2.8 mm)
- Narrow angle, i.e. < Schaefer grade 2
- Congenital eye abnormality
- Pigment dispersion syndrome
- Pseudophakic patients with malpositioned, subluxated or unstable capsular fixated intraocular lens
- Inability to achieve secure placement in the designated location e.g. due to absence of a secure peripheral anterior capsule, absence of intact zonules, or irregular anatomy of the ciliary sulcus
- Active ocular diseases (chronic severe uveitis, proliferative diabetic retinopathy, chronic glaucoma not responsive to medication, iris atrophy, severe zonulopathy)
- Iris neovascularization
- Inadequate visualization of the fundus on preoperative examination
- Bacterial or viral endophthalmitis
- Choroidal hemorrhage
- Retinal detachment
- Pseudoexfoliative syndrome
- Corneal decompensation
- Irregular astigmatism
- Age-related Macular Degeneration and other progressive retinal degenerations
- Pathological pupil reactions
- Preoperative ineffective miotic pupillary reaction or non-mydriatic pupil size of more than 2.5 mm under photopic conditions
- Severe corneal dystrophy
- Diabetic retinopathy
- Clinically significant macular or Retinal Pigment Epithelium changes
- Previous retinal detachment

- Pregnancy

TRIFOCAL MODELS

- Patients with a multifocal primary IOL in the capsular bag
- Keratoconus
- Any eye disease in which postoperative visual acuity is not expected to be better than 0.5 (e.g. amblyopia, nystagmus, retinitis pigmentosa, aniridia, eccentric pupil)
- Monocular patients
- Individuals who drive at night for a living or whose occupation or hobbies depend on good night vision
- Individuals who need good near vision in semidarkness

TORIC MODELS

- In case of patients who underwent previous refractive treatment – for example any kind of keratoplasty – the indication should be determined very carefully

SCHARIOTH MACULA LENS MODEL

- Active neovascular (wet) age-related macular degeneration

COMPLICATIONS

As with any surgical procedure, there is risk involved. The following non-exhaustive list specifies the complications that have been associated with the implantation of Intraocular lenses:

Disease related complications

- Corneal damage or edema
- Secondary glaucoma

Preoperative

- Pupillary block
- Iris trauma
- Hemorrhage

Postoperative

- Intraocular infection
- IOL exchange or extraction
- Uveitis
- Cystoid macular edema
- Postoperative opacification/calcification of the IOL
- TASS, Endophthalmitis
- Asthenopic discomfort, adaptational difficulties
- Reduced contrast sensitivity
- Perception of halos or radial lines around point sources of light
- Dissatisfactory visual outcome due to incorrect IOL refraction
- Reduced vision at night or in poor visibility conditions
- Acceleration of the preexisting Macular degeneration leading to blindness.

WARNINGS

- 1stQ Addon Intraocular lenses are designed to be implanted into the ciliary sulcus in the posterior chamber. There is no sufficient clinical data demonstrating the safety and efficacy of

an implantation in the capsular bag.

- Examine the package labels carefully for information about the lens model, power and expiration date. Lenses should not be used after the expiration date.
- DO NOT resterilize or reuse the lens by any method.
- DO NOT USE the IOL if the packaging is damaged or wet and lens sterility may have been compromised.
- DO NOT USE the product if the package was unintentionally opened before use.
- Store the unopened IOL box in a dry place, away from moisture and direct sunlight at 15-35°C.
- DO NOT USE hydrophilic IOLs if there is no fluid in the lens container.
- The storage fluid must not be used.
- A temporary opaqueness of the lens may occur in case of a considerable change of temperature. This phenomenon does not damage the lens material and the lens reverts to transparency after some time.
- A high level of surgical skills is required to implant intraocular lenses. The ophthalmic surgeon should have observed and/or assisted at numerous implantations and successfully completed one or more courses on IOL implantation before attempting to implant intraocular lenses.
- IOLs should be handled carefully to avoid damage to the lens optics or haptics. Non-toothed, polished instruments should be used, without grasping the optical area with forceps.
- Patients should be advised that unexpected outcomes may necessitate additional surgical intervention.
- Patient should be advised to wear spectacles in the sunlight to avoid damage by UV light.
- For optimal results, aim to achieve perfect IOL centration.
- Do not implant the supplementary (AddOn) IOL into the capsular bag.
- The product or its waste material should be disposed of in accordance with local/national regulations and requirements.
- Use of intraocular gas/air tamponade: Deterioration in transparency of the IOL has been observed upon the intraocular administration of SF₆ or C₃F₈ gases. Visually significant haze may develop, potentially leading to an IOL exchange.
- Carefully remove all viscoelastic material from both sides of the lens. Residual viscoelastic material may cause complications including increase of intraocular pressure.
- Patients with chronic autoimmune diseases under long term treatment should be considered as risk patients since exacerbation of their condition might occur.

TORIC MODELS

- Prior to surgery mark the operative eye with at least two reference points (patient in sitting position) or use an operating microscope that provides an axis guide.
- For optimal results, the surgeon must ensure the correct placement and orientation of the lens within the ciliary sulcus. The posterior surface of the IOL is marked with 2 linear indentations at the optic-haptic junctions that identify the flat meridian of the IOL. The cylinder axis marks should be aligned with the post-incision steep corneal meridian.
- Carefully remove all viscoelastic material from both sides of the lens. Residual viscoelastic material may cause complications including lens rotation resulting in the misalignment of the IOL, which compromises astigmatic correction.

TRIFOCAL MODELS

- Manage patient selection and operative technique carefully to ensure that the total

postoperative corneal astigmatism does not exceed 0.75 diopters.

- Only patients with fully functional pupil should be implanted.
- Some patients may experience reduced contrast sensitivity as compared to monofocal IOLs.
- Some patients may experience visual effects with the trifocal IOLs because of the superpositioning of focused and unfocused images. Visual effects may include the perception of halos or radial lines around point light sources under low illumination conditions.
- Patients should be advised that unexpected outcomes could lead to continued spectacle dependence.

DIRECTIONS FOR USE

1. Open the outer package to remove the protective peel-pouch or blister pack and verify that the IOL container information is consistent with the outer package labeling (e.g. power, model, SN).
2. Open the protective peel-pouch or blister and remove the lens container from the packaging in a sterile environment. Carefully open the cap and remove the lens holder from the fluid.
3. Transfer the lens, using sterile equipment to an appropriate loading device. Rinse the IOL with sterile Balanced Salt Solution. For loading and injection of the lens follow the Instructions for Use of the injector.
4. Unlike capsular bag implanted lenses, this 1stQ AddOn Intraocular lenses has to be folded in the opposite direction. Position the lens in the loading bay of the cartridge with the haptics positioned securely under the edge of the two grooves of the cartridge in a "reverse-U" (\cap) configuration. This ensures that the lens is folded and bent with the haptics pointing down. This way the lens will unfold with the leading haptics downwards into the ciliary sulcus.
5. Various surgical procedures can be utilized. The surgeon should select a technique that is appropriate for the patient.
6. Hydrophilic IOL should not be kept in open air for longer than 1 minute.

PREOPERATIVE CALCULATION OF IOL POWER

IOL power should be determined preoperatively based on proper biometry data using the formulae available in the literature. It is advised that surgeons personalize their calculation based on their surgical techniques, equipment and post-operative results. For 1stQ AddOn Intraocular lenses the use of a computerized/web-based Addon IOL calculator is highly recommended to ensure the best optical outcome. For further information please refer to:

<https://www.1stq.de/en/34-addoncalculator> or <http://www.1stq.de>

For diffractive trifocal lenses target emmetropia.


LIABILITY

1stQ does not bear any responsibility for improper model selection by the physician, for improper handling, use, surgical technique applied or for any other iatrogenic error caused by the implanting surgeon.

IMPLANT CARD AND PATIENT INFORMATION

One of the self-adhesive labels with the IOL data and UDI 2D barcode printed on it is designed to be placed on the Implant Card, also enclosed in the packaging. This Patient Card should be handed over to the patient for future reference allowing the patient to identify the surgeon and the type of IOL implanted.


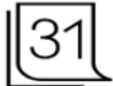


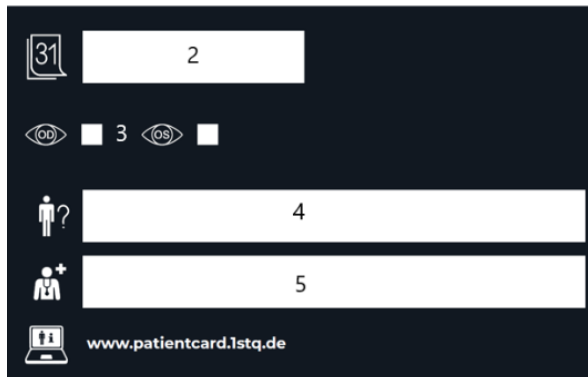





The implant card has to be filled in by the healthcare facility / healthcare provider as follows:




















1. Stick the label with UDI 2D barcode on the Implant card.
2. Fill in the date of implantation
3. Mark the implanted eye - left (OS) or right (OD).
4. Fill in the name of patient or patient ID.
5. Fill in the name and address of the healthcare institution / provider.
6. Fill in the device name.

The link to access the patient information is printed on the implant card.

SYMBOLS – IMPLANT CARD
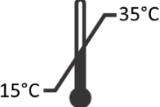
| | | | | | |
|---|--------------------------------------|---|----------------------------------|---|--|
|  | Patient Name or patient ID |  | Date of implantation |  | Name and Address of the implanting healthcare institution/provider |
|  | Name and Address of the manufacturer |  | Information website for patients |  | Device Name |
|  | Serial Number |  | Unique Device Identifier |  | Right Eye |
|  | Left Eye | | | | |

SYMBOLS – PACKAGING

| | | |
|--|--|--|
|  CE certified |  Keep dry |  Do not re-use |
|  Keep away from sunlight |  Consult instructions for use |  Do not re-sterilize |
|  Serial Number |  Use by date |  Sterilized using steam or dry heat |
|  Do not use if package is damaged |  Manufacturer |  Single sterile barrier system with protective packaging inside |
|  Temperature limit |  Date of manufacture |  Caution |
|  Medical device |  Unique Device Identifier | |

MANUFACTURER

1stQ GmbH
Konrad-Zuse-Ring 23
68163 Mannheim
GERMANY

Tel: +49 621 7176330
Fax: +49 621 7176333
www.1stq.eu
info@1stq.de

Any adverse events that the lens may have caused, any serious incident should be reported to 1stQ's Quality Assurance at info@1stq.de and to the competent regulatory authority.

LAST UPDATE: July 2021 Revision number: 01

This document is executed in the English language. In the event of any inconsistencies, the English version shall prevail.